Information

Informations • Informationen
Información • Informação • Bilgi
Informatie • Informazioni • informatsioon
informacija • informācija • информация
informacja • 信息 • מידע • معلومات

Distributor • Distributeur • Verteiler • Distribuidor • Distributör
• verdeler • Distributore • jagaja • skirstytuvass • izplatītājs •
Распределитель • Distribütör • 经销商 • תוארה • الموزع

Skins Sexual Health, Unit 5, Brunel Business Park,
Jessop Close, Newark, Nottinghamshire, NG24 2AG UK.

Advena Ltd, Tower Business Centre, 2nd
Floor, Tower St, Swatara, BKR 4013, Malta

Manufacturer • Fabricant • Hersteller • Fabricante • Tillverkare
• Fabrikant • fabbricante • tootja • gamintojas • ražotājs •
производитель • Üretici firma • 生产厂家 • יצרן • الشركة المصنعة

Dalian Latex Co., Ltd. No 18 Haixing Road
Pulandian District, Dalian, Liaoning Province, China.

Skins Condoms comply to the ENISO 4074
standard. Instructions revised: 10/2023
Expiry date of a condom is 5 years

SKCL-V2-10-2023

Read the leaflet before use

Medicines that might come in contact with the condom may damage it. Ask your doctor or pharmacist for advice. Only use additional lubricant that is safe to use with latex condoms. Oil-based products can damage a condom. No contraceptive can guarantee 100% protection against pregnancy. HIV or Sexual Transmitted Infections. Made from natural rubber latex. The use of latex might cause an allergic reaction including anaphylactic shock. Should the condom tear or leak during use, seek medical advice within 72 hours. Only use a condom from undamaged packaging. Never use a condom from a damaged foil. Stop and check if you feel the condom is slipping off. Also check when the condom feels too tight to prevent tearing. Only use a condom once. Multiple use will increase the risk of tearing and infections. Store in a cool dry place away from sunlight.

Instructions

- 1) Tear open the foil and take the condom out. Be careful with sharp fingernails or jewellery.
- 2) Squeeze the tip between your thumb and your index finger to expel the air. Put the condom on when the penis is erect and before any physical contact, to prevent pregnancy or transmitting STIs.
- 3) If you accidentally roll the condom down the wrong way, throw it away and use a new one.
- 4) After ejaculation, withdraw whilst the penis is still erect. Hold on to the base of the condom in order to prevent it sliding off.
- 5) Tie a knot in the condom and throw it away. Never flush it down the toilet.

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GB Extra Large Condoms Parallel sided transparent lubricated condom. Contains Natural Rubber Latex. Store in a cool dry place away from sunlight. No contraceptive can guarantee 100% protection against pregnancy, STI's or the HIV virus. Do not reuse. Consult instructions for use. The use of latex might cause an allergic reaction including anaphylactic shock.

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Skins Sexual Health Ltd is a British registered company. Skins and the Skins logo are registered trademarks of Skins Sexual Health Ltd based in the United Kingdom. Skins Condoms are made to the highest European quality standards and comply with International Standard **ENISO 4074**

EC REP

CE Partner 4U B.V
ESDOORNLAN 13, 3951 DB
Maan, The Netherlands

Distributor: Skins Sexual Health Ltd,
Brunel Business Park, Newark,
Nottinghamshire NG24 2AG, UK.

SKINS CUBE: 16 EXTRA LARGE CONDOMS

Contents: 16 Skins Condoms. Transparent natural, lubricated condom, parallel sided with reservoir. Vanilla fragranced. Made of natural rubber latex.

W:57±2mm L:≥205mm

Read instructions prior to use. Oil based lubricants or massage oil could damage the condom. Some topical medications might affect condoms on contact. Please consult your doctor or pharmacist prior to use. Only use condoms from undamaged packaging. Seek medical advice within 72 hours if the condom tears or leaks during use. Multiple use will increase the risk of tearing and infections. The use of latex might cause an allergic reaction including anaphylactic shock. No method of contraception can provide 100% protection against pregnancy or the transmission of HIV or other STI's.

Please read the leaflet
carefully before use.
Single Use Only

REF SKXL16


Customer Information:
www.skinscondoms.com
+44 1636 593 943

Dalian Latex Co., Ltd. No 18
Haixing Rd, Pulandian District,
Dalian, Liaoning Province, China

MD

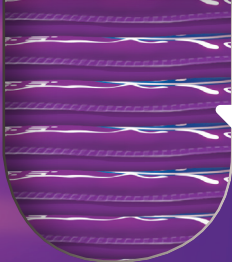THIS
IS WHAT'S
INSIDE THE
CUBE...

CE 0197

ENISO
4074

Our Mission

Skins understand that sex is a fundamental part of physical and emotional wellbeing. We believe that a safe, healthy and rewarding sex life should be everyone's to enjoy. Our aim is to understand what 'better sex' means to you and our promise is to provide the information.

Ultimate Feeling

Skins are thinner with more lubricant than most standard condoms giving you the most intimate skin on skin feeling possible whilst staying better protected.