

## **Preparation and Usage**

### *Before Use*

- Only use lubricants recommended for use with condoms. For anal sex use additional lubrication (do not use Durex Tingling or Embrace) on the outside of the condom.
- Before you use this condom, check the expiry date on the condom wrapper.
- If the individual container is obviously damaged, throw that one away and use a new one from an undamaged package.
- To help you get it on right first time, the Durex logo on the foil shows you the bottom of the condom.
- Put a condom on before the penis touches the other person. This helps to prevent pregnancy and the possibility of catching sexually transmitted infections during vaginal, anal or oral sex.

### *How to Use Your Condom*

- 1) Tear the wrapper open from the serrated edge. Be careful with the condom, it could get damaged by fingernails or sharp objects.
- 2) With the roll outside, squeeze the teat and place the condom on the tip of the erect penis.
- 3) Roll down to the base. If the condom doesn't unroll, it may be on backwards, discard and use a new one. Stop and check if you feel the condom slipping off or too tight on the penis because this might lead to breakage. You should take the condom off soon after ejaculation. Hold the condom firmly at the base of the penis before pulling out
- 4) Throw the foil and used condom into a bin. Don't flush down the toilet.

## **Storage**

- Keep cool.
- Keep away from sunlight.

## **Safety Information**

- Perfoma™ silicone lubricant contains 5% benzocaine.
- Condoms may present a choking hazard.
- Keep out of reach of children.
- Only use a condom once.
- Reuse may increase risk of failure or infection.
- If you use the condom for oral sex first, you should use a new condom for any other type of sex that follows.
- Only use lubricants recommended for use with condoms. Oil-based ones (petroleum jelly, baby oil and some pessaries) and some topical medicines that go on the penis or vagina can damage condoms.
- Speak to your doctor or pharmacist if you are unsure if you are allergic to latex or if irritation or discomfort persists after use or you are applying topical medicines to your penis or vagina.
- If a condom leaks or bursts during sex consult a pharmacist or doctor as soon as possible and within 72 hours. Serious incidents must be reported to Reckitt and the local competent authority who will provide further advice.
- This condom is made of natural rubber latex. If you have a known allergy to latex, do not use this condom as this may cause allergic reactions including anaphylactic shock.
- Stop use immediately if you feel discomfort or irritation while using condoms.
- Remember no method of contraception works 100% against pregnancy, HIV or sexually transmitted infections.
- Contains a medicinal substance.