6B Important information for PASANTE Natural Rubber Latex Condoms, please read carefully. Always handle the condom carefully including removal from the package so as to avoid damage to the condom by finger nails, jewellery, etc. If condom foil is damaged discard the condom and use a new undamaged one. Condoms made from natural rubber are not compatible with oil based lubricants, including baby oil. A condom should never be used for a second time, condoms are for single use and if reused there might be an increased risk of failure or infection, the risk of breakage is also higher. The use of a condom cannot guarantee 100% protection against pregnancy, HIV, or Sexually Transmitted Infections (STIs). If in doubt always consult a doctor. If additional lubricant is desired, use the type of lubricant recommended on the condom. Use of oil based lubricants on a latex condom, such as water-based lubricant such as Pasante Light Lubr. Avoid the use of oil based lubricants, such as petroleum jelly, baby oil, body lotions, massage oils, butter, margarine, etc., as these are deleterious to the integrity of the latex condom. Use of condom during anal sex increases the risk of breakage. It is recommended to use additional lubricant if vaginal lubrication is insufficient or for anal sex to minimize the risk of condom breaking or coming off. Consult a doctor or pharmacist about the compatibility of any medication with the use of condoms. Carry condoms in your wallet, back pocket, handbag, or similar is not advisable. If you have unprotected sex or your method of contraception has failed, contact your Sexual Health Clinic or GP for emergency contraception advice preferably no later than 72 hours. Keep away from children. Children can choke or suffocate on uninflated or broken condom fragments. Use of condom during oral sex increases the risk of asphyxiation, choking or oral infection. A latex condom is a medical device and its intended use is by adults for sexual activity. Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

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[illegible][illegible]

eista, lš
 rgines
 okite
 udojant
 didesnē
 edicā

The image shows a portion of a document with a large, dark blue, rounded rectangular logo on the left. Inside the logo, the word "Pasante" is written in a large, white, serif font. Above the logo, there is a small registered trademark symbol (®). To the right of the logo, the text "Instructions For Use" is written in a large, bold, black, sans-serif font, followed by "for Condoms" in a slightly smaller, regular, black, sans-serif font. Below the logo, there is a horizontal dashed line. To the right of the dashed line, the text "Pasante Healthcare Ltd." is written in a black, sans-serif font, followed by "Commerce Way", "Lancing, West Sussex", "BN15 8TA", "UK", and "+44 (0) 1903 753844" on separate lines. At the bottom right, the website "www.pasante.com" is written in a black, sans-serif font.