Pasante® Vanilla Dam

contains one latex dam

6B Instructions for use

- Tear open the package carefully. Do not use fingernails, teeth or anything that can damage the dam. Remove the dam from package.
- For oravaginal use, lay the dam flat on the entire vulva, making sure to cover the vaginal opening and the clitoris. For oral/anal use, use a dam to cover the entire anus. Hold the edges of the dam during use to ensure it stays in place.
- Optional: Moisten the vulva or anal area with water-based lubricant (such as Pasante Light Lube) before putting the dam in place. Be sure to remove the lubricant from your fingers as this may make it difficult to hold the dam in place.
- Always use a new dam if you switch from orallanal to oravaginal sex to reduce the risk of infection from harmful anal germs.

Warnings

- Do not use during penetrating penile/vaginal or penile/anal intercourse.
- If a lubricant is required, use only lubricants claimed as compatible with natural latex rubber film. DO NOT USE OIL-BASED LUBRICANTS, such as those made with petroleum jelly, baby oil, body lotions, massage oils, butter, margarine, as these can damage the dam.
- Do not stretch the dam.
- Store in a cool, dry place. Do not expose to direct sunlight. Heat and light accelerate the degradation of latex.
- Dispose after single use. Do not reuse or reuse dams.
- This product contains Natural Rubber Latex which may cause allergic reactions. If irritation or discomfort occurs, discontinue use and consult a doctor.
- Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

1	CONSULT INSTRUCTIONS FOR USE	2	DO NOT REUSE
LOT	BATCH CODE		KEEP DRY
	USE BY	REF	CATALOGUE NUMBER
	CONTAINS OR PRESENCE OF NATURAL RUBBER LATEX		KEEP AWAY FROM SUNLIGHT
	RECYCLABLE PACKAGE		DO NOT USE IF PACKAGE IS DAMAGED
CE REP	AUTHORISED REPRESENTATIVE IN THE EUROPEAN UNION	MANUFACTURER	IMPORTER
CE 0123	EUROPEAN CONFORMITY	MD	MEDICAL DEVICE
		UDI	UNIQUE DEVICE IDENTIFICATION

Prophylactic dam is a medical device to be used by both persons – Adult male or female. Pasante dams are thin, latex sheet designed as a barrier for use while performing oravaginal sex or oral/anal sex. When properly used, Pasante dams may help to reduce the risk of transmission of many STIs, including Syphilis, Gonorrhoea, Chlamydia, Genital Herpes and HIV. However they cannot eliminate 100% risk. During intimate contact, lesions and various body fluids can transmit STIs, therefore, a Pasante dam should be applied each time prior to sexual contact.

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Kaip u dėti oraliniam / vaginaliniam seksui

Prophylactic dam is a medical device. Pasante dams are designed as a barrier for use while performing oravaginal sex or orallanal sex. When properly used, Pasante dams may help to reduce the risk of transmission of many STIs, including Syphilis, Gonorrhoea, Chlamydia, Genital Herpes and HIV. However they cannot eliminate 100% risk. During intimate contact, lesions and various body fluids can transmit STIs, therefore, a Pasante dam should be applied each and every time prior to sexual contact.

Contains one 10" x 6" latex sheet.

CAUTION: This product contains Natural Rubber Latex which may cause allergic reactions. If you are allergic to natural rubber latex products, consult your doctor for advice before use. For maximum benefit, carefully follow the instructions inside this package. You may need to consult a doctor, or pharmacist about the compatibility of topical medicines that could come in contact with the dam.

Store in a cool dry place away from direct sunlight.

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