Information

Informations • Informationen
Información • Informação • Bilgi
Informatie • Informazioni • informatsioon
informacija • informācija • информация
informacja • 信息 • מידע • معلومات

Distributor • Distributeur • Verteiler • Distribuidor • Distributör
• verdeler • Distributore • jagaja • skirstytuvass • izplatītājs •
Распределитель • Distribütör • 经销商 • תוארה • الموزع

Skins Sexual Health, Unit 5, Brunel Business Park,
Jessop Close, Newark, Nottinghamshire, NG24 2AG UK.

CE REP CE Partner 4U B.V. ESDOORNLAN 13,
3951 DB Moon, The Netherlands

Manufacturer • Fabricant • Hersteller • Fabricante • Tillverkare
• Fabrikant • fabbricante • tootja • gamintojas • ražotājs •
производитель • Üretici firma • 生产厂家 • יצרן • الشركة المصنعة

Dallan Latex Co., Ltd. No 18 Haixing Road
Pulandian District, Dalian, Liaoning Province, China.

Skins Condoms comply to the ENISO 4074
standard. Instructions revised: 11/2023
Expiry date of a condom is 5 years

SKCL-V2-11-2023

Read the leaflet before use

Medicines that might come in contact with the condom may damage it. Ask your doctor or pharmacist for advice. Only use additional lubricant that is safe to use with latex condoms. Oil-based products can damage a condom. No contraceptive can guarantee 100% protection against pregnancy. HIV or Sexual Transmitted Infections. Made from natural rubber latex. The use of latex might cause an allergic reaction including anaphylactic shock. Should the condom tear or leak during use, seek medical advice within 72 hours. Only use a condom from undamaged packaging. Never use a condom from a damaged foil. Stop and check if you feel the condom is slipping off. Also check when the condom feels too tight to prevent tearing. Only use a condom once. Multiple use will increase the risk of tearing and infections. Store in a cool dry place away from sunlight.

Instructions

1) Tear open the foil and take the condom out. Be careful with sharp fingernails or jewellery.

2) Squeeze the tip between your thumb and your index finger to expel the air.

Put the condom on when the penis is erect and before any physical contact, to prevent pregnancy or transmitting STIs.

If you accidentally roll the condom down the wrong way, throw it away and use a new one.

3) After ejaculation, withdraw whilst the penis is still erect. Hold on to the base of the condom in order to prevent it sliding off.

4) Tie a knot in the condom and throw it away. Never flush it down the toilet.

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