
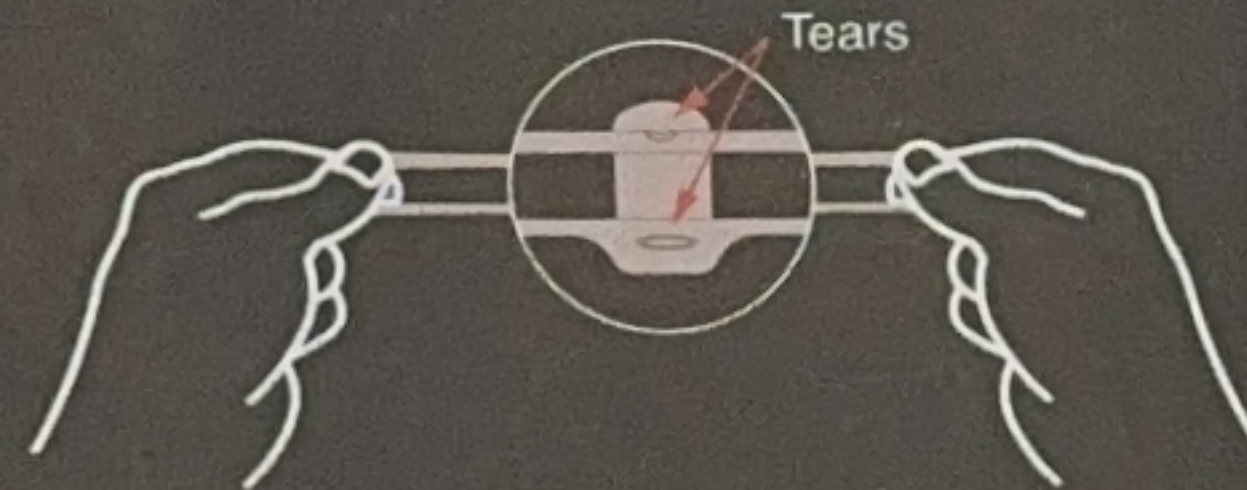


PRECAUTIONS

Over-stretching the ring, as in this image, can tear the soft, proprietary elastomer, which has been formulated for your comfort - unlike hard silicone. This is not a rubber band!

In rare cases when the device is too tight or has been left on for an extended period of time, blood vessels may bruise. While these should resolve quickly, in the very unlikely event that they do not, please contact a health care provider.

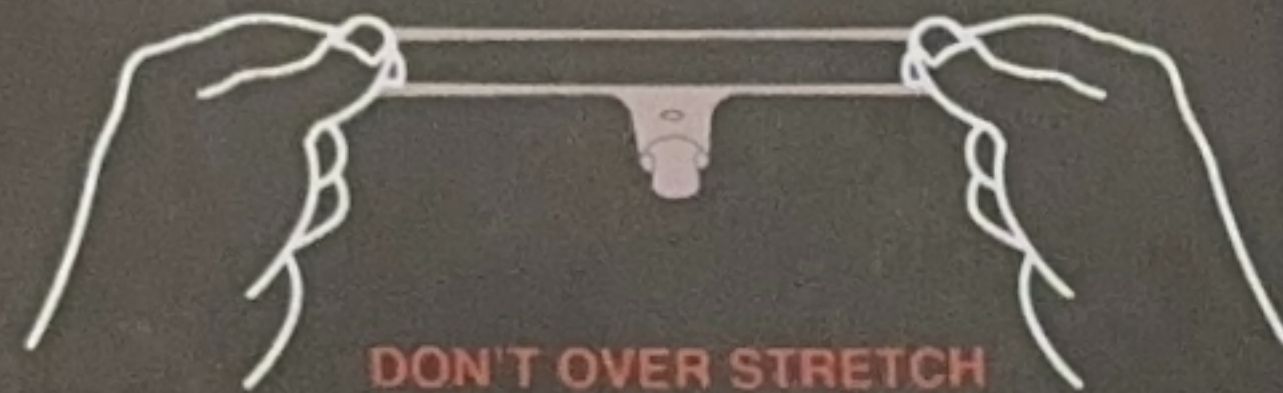

PRODUCT WARRANTY

Please inspect your product when it arrives. FirmTech will replace or refund any damage caused by a defect in our material or workmanship. This warranty does not cover normal wear and tear caused by handling, non-consumable parts such as rechargeable batteries which naturally have a limited lifespan and declining capacity over time. If you receive a product and it does not fit or it's not what you expected it to be, it may be returned in certain circumstances. For more information please visit our website's return policy.

CARE AND CLEANING

Clean ring with mild soap and water. We recommend using water-based lubricants for sexual activity, as frequent use of silicone-based lubricants may damage the ring.

Warning

Do not use this device if:

- A) You have urinary tract infections (UTI) sexually transmitted diseases (STDs) or prostatitis
 - B) You have urine/bladder retention or distention
 - C) You have renal/kidney failure
 - D) You suffer from Peyronies disease, urethral strictures or priapism history
 - E) You have decreased sensation for pain in the area of the penis
 - F) You are under the influence of drugs or alcohol
 - G) You cannot get even a partial erection under any circumstances
 - H) You have latex allergies or sensitivities.
- Constriction rings do not prevent pregnancy

WE ADVISE AGAINST USING THE DEVICE FOR MORE THAN 1 HOUR

Consult a health care provider if any complications occur and discontinue use of device.