

The intended use of this product is for contraception and for the prevention of the transmission of sexually transmitted diseases. The device is intended to be used by the sexually active male population, but through use, can come into contact with the female population.

BEFORE YOU GET GOING

Before you use this condom, check the expiry date on the condom wrapper. Do not use the condom if the materials are sticky or brittle. If the individual container is obviously damaged, throw that one away and use a new one from an undamaged package. Put a condom on before the penis touches the other person. This helps to prevent pregnancy and the possibility of catching sexually transmitted infections during sex.

HOW TO USE YOUR CONDOM

- Carefully tear off foil pack, making sure not to damage the condom. Avoid damage to the condom by fingernails or sharp objects.
- With the roll outside, squeeze the teat and place the condom on the tip of the erect penis.
- Roll down to the base. If the condom doesn't unroll, it may be on backwards, discard and use a new one. Stop and check if you feel the condom slipping off or too tight on the penis because this might lead to breakage. You should take the condom off soon after ejaculation. Hold the condom firmly at the base of the penis before pulling out.
- Throw the foil and used condom into a bin. Don't flush down the toilet.



JUST A FEW WARNINGS

Only use silicone oil-based lubricants. **DO NOT USE WATER BASED LUBRICANTS**, as it may damage the condom and cause subsequent failure of the product. Some topical medicines that go on the penis or vagina can damage condoms.

STOP USE: If you feel discomfort or irritation while using condoms. Only use a condom once. Reuse may increase risk of failure or infection. Condoms may present a choking hazard. Keep out of reach of children.

WHEN TO SEEK ADVICE

Speak to your doctor or pharmacist if you experience allergic reaction or if irritation or discomfort persists after use or you are applying topical medicines to your penis or vagina. If a condom leaks or bursts during sex consult a pharmacist or doctor as soon as possible and within 72 hours. Serious incidents must be reported to Reckitt and the local competent authority who will provide further advice.

Consult instructions for use	Do not re-use	Does not contain or no presence of natural rubber latex	Do not use if package is damaged	Keep dry	Keep away from sunlight	Medical device	Manufacturer
EC REP	Importer	Distributor	Expiry date	Batch code			
Authorized representative in the European Community/ European Union							